



Instructions for hair sampling


Prearrangement



Required materials:

- ✓ Aluminum foil
- ✓ Scissor
- ✓ Paper clip
- ✓ Hairgrip (included in delivery)
- ✓ Comb (included in delivery)
- ✓ Packthread (included in delivery)

Step 1

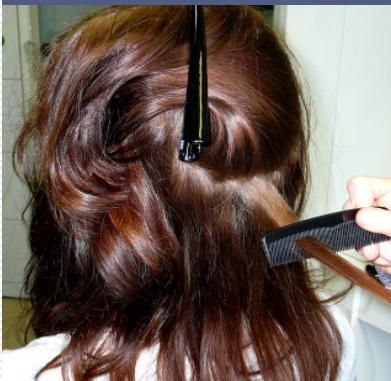

- Separate the hair with the hairgrip at the back of the head
- Separate a strand of hair (**thickness of a pencil**) ca. 2 cm below the cranial bone and comb through
- Alternatively separate several fine strands at the same spot

Step 2



- Thread the combed strand **through the loop** and tie up

Step 3



- Cut hair **as close as possible to the scalp**

Step 4



- Put the strand(s) into aluminum foil
- Sign the part which is close to hair root with a permanent marker

Step 5



- Fold the aluminum foil
- If the sample is longer than aluminum foil you can cut hair tips at the lower end

Step 6



- Fix the part of the hair which is close to hair root with a paper clip
- Do **not fold** the sample
- Sign the sample clear
- Store the sample try and in the dark